

SURE CHECK® HIV 1/2 Assay

A unique, easy-to-use, self-contained collection and testing device for the rapid, visual detection of antibodies to HIV-1 and HIV-2.

Simple

- CLIA waived¹
- Sample collection and test device in one
- 2-step test procedure
- Room temperature storage: 46° to 86°F (8° to 30°C)



Reliable

- FDA approved
- Sensitivity: 99.7%; Specificity: 99.9%²
- Built-in IgG procedural control

Fast

- Rapid results: 15 minutes

Smallest sample size: 2.5 µL



3661 Horseblock Road, Medford, NY 11763 USA
1-844-CHEMBIO (1-844-243-6246)
CustomerService@chembio.com

 **CHEMBIO**
DIAGNOSTIC SYSTEMS, INC.
Rapid Tests for Earlier Treatment™


¹ CLIA waived for fingerstick and venous whole blood
² See product insert for complete performance details

Manufactured in the USA

2-Step Procedure*



1 ADD SAMPLE

Prick finger using lancet. TOUCH blood drop with capillary tip of sampler.

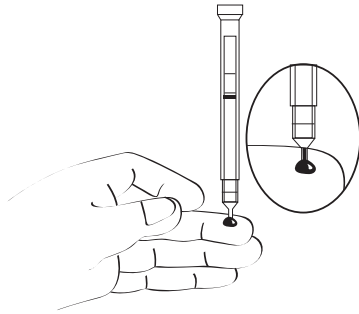


2 RUN TEST

PUSH capillary tip of sampler completely into buffer vial. Read the test between 15 and 20 minutes.

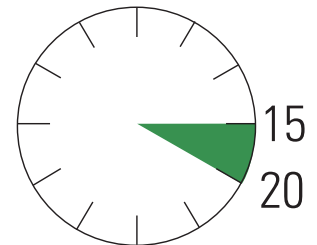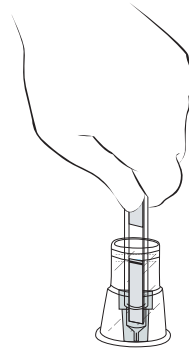

Possible Results

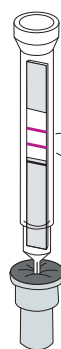

NON-REACTIVE




1 line

REACTIVE




2 lines

INVALID

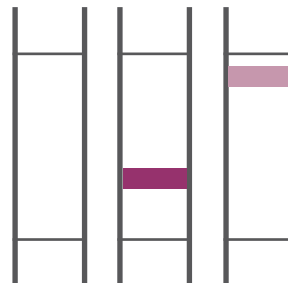

No control line, or
outside area

INVALID

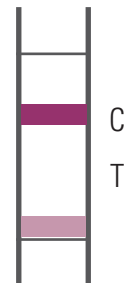

Test line
outside area

* Before testing, please refer to the complete product insert provided in the test kit

Ordering Information

Reimbursement Code: 86703-QW, G0433-QW

Description	Contents	Catalog Number
SURE CHECK® HIV 1/2 Assay FDA Approved, CLIA Waived/Non-Waived	25 Individually Pouched Test Devices (Including a Sterile Lancet, Bandage, and Desiccant Packet) 25 Disposable Test Stands 25 Copies of Subject Information Notice 1 Quick Reference Instructions (for CLIA Waived) 1 Product Insert	60-9507-0
HIV Rapid Test Control Pack FDA Approved	One Pack Contains: - 1 HIV-1 Reactive Control 0.25 mL - 1 HIV-2 Reactive Control 0.25 mL - 1 HIV Nonreactive Control 0.25 mL Each kit control vial contains sufficient volume to run up to 100 tests	60-9549-0

SURE CHECK® HIV 1/2 Assay is sold in the EU as STAT-VIEW® HIV 1/2 Assay