

NEXT GENERATION DPP®

# DPP® HIV 1/2 Assay

A rapid, qualitative, point-of-care assay for the detection of HIV-1 and HIV-2 antibodies in oral fluid, fingerstick or venous whole blood, serum or plasma.

## Simple

- CLIA waived<sup>1</sup>
- Oral fluid or blood samples
- Ready-to-use reagents
- Store at 36° to 86°F (2° to 30°C)


## Reliable

- FDA approved
- Excellent sensitivity and specificity<sup>2</sup>
- Built-in IgG procedural control

## Fast

- Results in as little as 10 minutes

Distinct Test and Control Lines

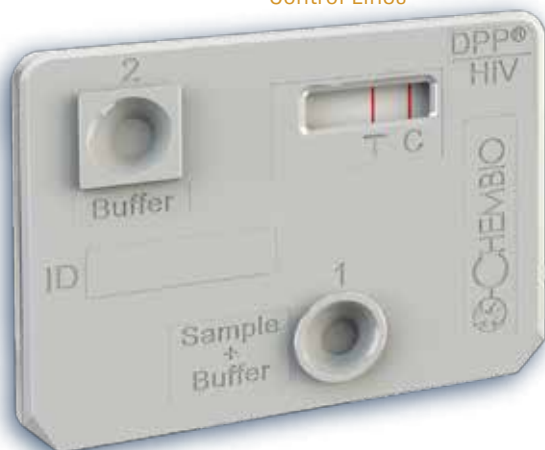

## Oral fluid or blood assay



Soft Foam Tip

3661 Horseblock Road, Medford, NY 11763 USA  
1-844-CHEMBIO (1-844-243-6246)  
CustomerService@chembio.com


 **CHEMBIO**  
DIAGNOSTIC SYSTEMS, INC.  
*Rapid Tests for Earlier Treatment™*

Manufactured in the USA


# Administering Test\*

**1** OBTAIN oral fluid sample by swabbing around the outer gums for a minimum of 30 seconds.

Or OBTAIN a blood sample with sample loop.


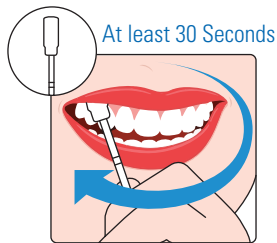


OR

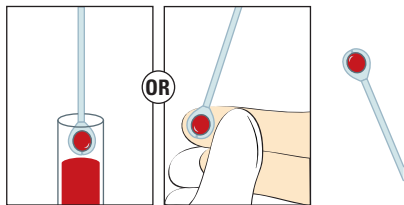


**2** INSERT the oral fluid swab into the bottom of the SampleTainer® bottle. SNAP & DETACH leaving swab in vial. Replace black cap. SHAKE SampleTainer® bottle 10 sec.



Or INSERT the sample loop with collected blood sample into the bottom of the SampleTainer® bottle. BEND & TWIST to detach from loop end remaining in vial. REPLACE black cap. SHAKE SampleTainer® bottle 10 sec.

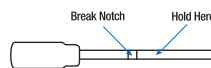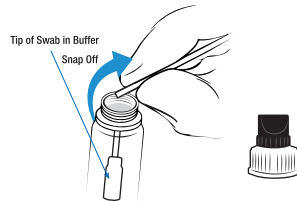



OR

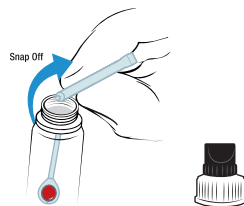

**3** REMOVE the black cap. ADD 2 drops into the Sample + Buffer well 1. WAIT 5 minutes.

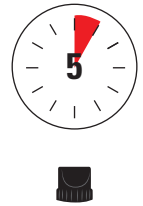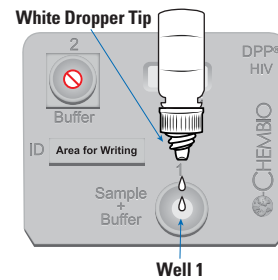

**4** ADD 4 drops of running buffer (green cap) to buffer well 2. **Blood:** Read result at 10 minutes after the addition of the running buffer to well 2; do not read results after 25 minutes. **Oral fluid:** read result between 25-40 minutes. Wait 40 minutes to confirm negative.

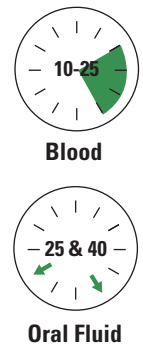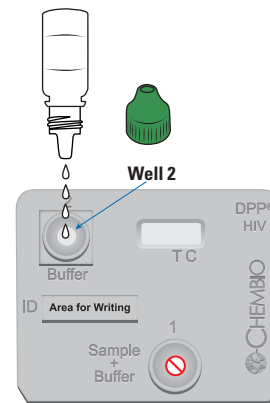


\* Before testing, please refer to the complete product insert provided in the test kit

## Interpretation of Results:

NON-REACTIVE



REACTIVE

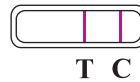

Description	Contents	Catalog Number
DPP® HIV 1/2 Assay, FDA Approved	20 Individually Pouched DPP HIV 1/2 Test Devices with desiccant pouch 1 Quick Reference Instructions (for CLIA Waived) 20 Copies of Subject Information Notice 20 Oral Fluid Swabs 20 Disposable 10µL Sample Loops 20 DPP HIV SampleTainer® Bottle – BLACK Cap 1 DPP HIV Running Buffer – GREEN Cap 1 Product Insert for the DPP HIV 1/2 Assay	65-9500-0
DPP® HIV 1/2 Rapid Test Control Pack	1 pack contains: - 1 HIV-1 Reactive Control 0.5 mL - 1 HIV-2 Reactive Control 0.5 mL - 1 HIV Nonreactive Control 0.5 mL Each kit control vial contains sufficient volume to run 50 tests	60-9552-0

<sup>1</sup> CLIA waived for oral fluid, fingerstick and venous whole blood

<sup>2</sup> Sensitivity: 99.9% Whole Blood/Plasma/Serum; 99.8% Fingerstick; 98.9% Oral Fluid  
Specificity: 100% Fingerstick; 99.9% Oral Fluid/Whole Blood/Plasma/Serum