

# STAT-VIEW<sup>®</sup> HIV 1/2 Assay

A unique, easy-to-use, self-contained, single-use collection and testing device for the rapid, visual detection of antibodies to HIV-1 and HIV-2.

## Convenient and Cost Effective

- Ideally suited for field and point-of-care testing
- Room temperature storage
- Long shelf life
- No special laboratory equipment required


## Time and Labor Saving

- Unique device that combines collection and testing device
- Ready-to-use reagents
- Simple procedure
- Rapid 15-minute results
- Results are easy to interpret

## Reliable Results

- Built-in IgG procedural control
- 100% sensitivity and 99.8% specificity\*

Minimal sample size: 2.5  $\mu$ L

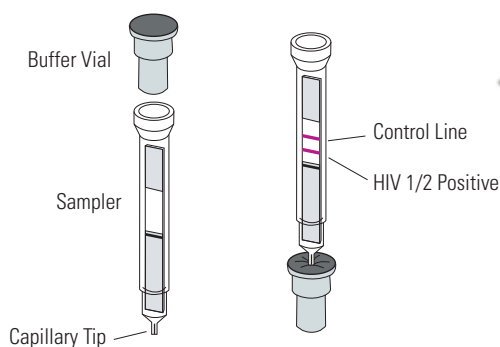

Manufactured in the USA

 **CHEMBIO**  
DIAGNOSTIC SYSTEMS, INC.  
*Rapid Tests for Earlier Treatment™*


\* See product insert for complete performance details

CE-Marked

## 2-Step Procedure\*



### 1 ADD SAMPLE

Prick finger using lancet. TOUCH blood drop with capillary tip of sampler.

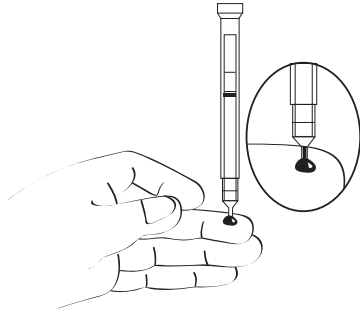

### 2 RUN TEST

PUSH capillary tip of sampler completely into buffer vial. Read the test between 15 and 20 minutes.

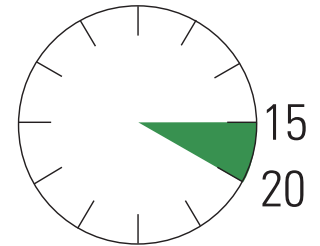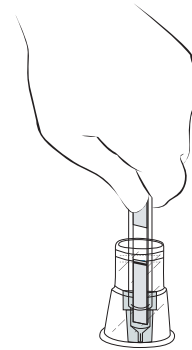

## Interpretation of Results


NEGATIVE



POSITIVE



INVALID

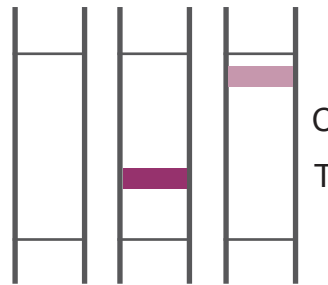

No control line, or  
outside area

INVALID



Test line  
outside area

\* Before testing, please refer to the complete product insert provided in the test kit

## Ordering Information

Description	Contents	Catalog Number
STAT-VIEW® HIV 1/2 Assay CE-Marked	25 Individually Pouched Test Devices (Including a Sterile Lancet, Bandage, and Desiccant Packet) 1 Disposable Rack 1 Product Insert	60-9520-0